130 mm

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Omnisept

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(GB)
Area of application:
For cleaning and disinfecting of rotating and surgical instruments.
Indications for use:
Solution is in a concentrate form which has to be diluted with water. Heavily contaminated instruments should first be cleaned before submerging into the disinfectant. After disinfectant, instruments are to be thoroughly rinsed with water and dried. Do not mix with other types of disinfectant or cleansers. Renew the solution depending on the degree of soiling (best daily). The instrument disinfection can be used in the defined concentration also in ultrasonic bath.
Microbiological Effectiveness:
Bactericide, levurocidal (Candida albicans), tuberculocide, "restricted viruses" (effective against membrane covered viruses, i.e. HBV, HIV, HCV).
Effectivity:
Bacteria in accordance to VAH/DGHM, EN 13727, EN 14561 (high bio load):
3% 15 min. 2% 30 min. 1% 60 min.
Levurocidal in accordance to VAH/DGHM and EN 13624, EN 14562 (high bio load):
3% 15 min. 2% 30 min. 1% 60 min.
TbB (Tuberculosis-inactivation in accordance to VAH/DGHM: 3% 60 min.
Enveloped viruses (i.e. HBV, HIV, HCV) in accordance to DVV/RKI (BVDV and vaccinia virus accordance EN 14476 with high bio load): 3% 15 min. 1% 60 min.

Ultrasonic bath (Bacteria and Candida albicans in accordance to VAH/DGHM: (high bio load): 3% 5 min.

Special hazards information:
Causes severe skin burns and eye damage. May be harmful if swallowed. Do not breathe dust/fume/gas/mist/vapours/spray. Wear protective gloves and eye protection. IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. IF INHALED: Remove to fresh air and keep at rest in a position comfortable for breathing. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Call a POISON CENTRE or doctor/physician if you feel unwell. If experiencing respiratory symptoms: Call a POISON CENTRE or doctor/physician. Store in a closed container. Dispose of contents/container in accordance with local/regional/national/international regulations. Contains piperazine. May produce an allergic reaction.

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