# Omnisept Plus

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GB

#### Properties:

Medical product fulfills the sophisticated requirements for disinfection performance and material compatibility of (surgical) instruments. Omnisept Plus shows activity that is bactericidal (including MRSA) and yeasticidal (EN 13727/EN 13624); it has been tested according to the guidelines of the German Society for Hygiene and Microbiology (DGHM, status: 1 September 2001) with short-time exposure limits of 2% in 15 minutes and 1% in 30 minutes under high organic load. Shows tuberculocidal activity (M. terrae) according to EN 14348 in 3% solution in 30 minutes and in 1% solution in 60 minutes under high organic load. Listed with the German Association for Applied Hygiene (VAH). Complete virus inactivation (HBV/HCV/HIV/BVDV/Vaccinia/HPV/Adeno/Polio) is achieved in a 3% solution in 15 minutes. Also effective against papovaviruses and BVDV viruses in 2% and rotaviruses/noroviruses in 3% solution, each in 15 minutes under high organic load (EN 14476). Free of aldehydes and phenols. Excellent corrosion behavior. Effective in an ultrasonic bath in a 3% solution after 5 minutes. No fixation of proteins. The working solution is biologically degradable. The drum can be recycled.

#### Application:

To produce a 1% working solution, add 990 ml of tap water into the tub and mix with 10 ml Omnisept Plus concentrate. Recommended water quality < 8.4° dH. Strongly contaminated instruments should be cleaned first with warm water and a soft brush. Place the used instruments in the solution and cover with the lid. Ensure complete wetting. Avoid bubble formation. Observe minimum exposure time. Afterwards, remove the instruments, remove running water and dry with a clean cloth. Before use in an ultrasonic bath: Cover with the lid. Do not remove lid until after cleaning and cooling. Refresh used solution daily (RKI guidelines). Observe the restrictions on use issued by the instruments manufacturer. Sterilization according to RKI guidelines.

#### Active ingredients:

100 g concentrate contain:  
8,0 g Benzalkoniumchloride  
4,0 g Dodecylidipropylentriamine

#### Precautions:

Wear protective gloves/protective clothing/eye protection/face protection. Wash hands thoroughly after handling. Do not eat, drink or smoke when using this product. Avoid formation of aerosol. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. Wash contaminated clothing before reuse. IF IN EYES: Rinse cautiously with water for at least 15 minutes. Remove contact lenses, if present and easy to do. Continue rinsing. IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. Call a POISON CENTER or doctor/physician if you feel unwell. IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing. Keep out of reach of children. Store locked up. Keep cool. Avoid direct sunlight. Avoid release to the environment. For professional use only. Indicate batch number (LOT No.) to identify product in case of any occurrence. Dispose product according to official regulations.

The batch number (or LOT #) should be indicated on every procedure that requires identification of the product. Dispose of according to regulatory requirements.

H332 Harmful if inhaled.  
H314 Causes severe skin burns and eye damage.  
H317 May cause an allergic skin reaction.  
H410 Very toxic to aquatic life with long lasting effects.

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