OMNI PRAXIS
DENT®

We care. You smile

Omnisuc

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Absauganlagendesinfektion Konzentrat
Disinfettante concentrato per impianti di aspirazione
Ontsmetting instrumenten afzuiginstallatie conc.
Środek w koncentracji do dezynfekcji instalacji odsysającej
Evacuation system disinfection concentrate
Désinfection des systèmes d'aspiration conc.
Konzentrad sugsystemrengöring
Desinfekce odsávacích zařízení, koncentrát

e 2,5 L

156 mm

REV. 12/04_01

(D)

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(GB)

Areas of application: High performance solution designed for cleaning and disinfecting of dental suction system and splutter.

Indications for use: The concentrate must be first diluted with water – to 1 to 3% concentration, (10 to 30 ml concentrate per 1 ltr. water). Approx. 1 ltr of solution should be used per application. The suction system should be regularly disinfected, one to two times daily. Do not mix with other disinfectants or cleansers.

Microbiological Effectiveness: Bacteria in accordance with VAH/DGHM EN 13727, EN 14561 and EN 1276. Candida albicans in accordance to VAH/DGHM

and EN 13624, EN 14562, EN 1650. Enveloped viruses (e.g. HBV, HIV, HCV) in accordance to BGA (RKI)/DVG.
Effectivity: Bacteria and Candida albicans in accordance to VAH/DGHM as Instrument disinfection (high bio load); 2,0% 5 min., 1,0% 60 min. Bacteria in accordance to EN 1276 (high bio load); 2,0% 5 min. Candida albicans in accordance to EN 1650 (high bio load); 2,0% 5 min. Bacteria in accordance to EN 13727 and EN 14561 (high bio load); 3,0% 2 min. Candida albicans in accordance to EN 13624 and EN 14562 (high bio load); 3,0% 2 min. Enveloped viruses (i.e. HBV, HIV, HCV) in accordance to BGA(RKI)/DVG (bio load); 2,0% 2 min.

100 g contains: 7.5 g Dimethyloctylammoniumchloride, 0.6 g Benzylmethyldodecylammoniumchloride

Safety instruction: H 315 Causes skin irritation. H 319 Causes serious eye irritation. Wash thoroughly after handling. Wear protective gloves and eye protection. IF ON SKIN: Wash with plenty of soap and water. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If skin irritation occurs: Get medical advice/attention. If eye irritation persists: Get medical advice/attention.

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